
SAFETY GUIDELINES - PLEASE READ BEFORE USE

- ⚠ WARNING:** **Important: Read and understand these instructions before installing or using the Walker Tray. Ensure that the Walker Tray is installed as described in the following paragraphs before use. If the Walker Tray is not properly installed, personal injury and/or property damage could result.**
- ⚠ WARNING:** **GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation or use of this product.**
- ⚠ WARNING:** **If components are damaged or missing, contact your Graham-Field Health Products dealer immediately. DO NOT use substitute parts.**
- ⚠ WARNING:** **Do not sit, push down, or lean on the Walker Tray; this could cause the Walker Tray to collapse, resulting in loss of support.**

WALKER TRAY INSTALLATION (REFER TO PICTURE)

- ⚠ WARNING:** **Before installing the Walker Tray, ensure that the walker is completely unfolded and locked in open position.**

1. With the Walker Tray in a collapsed position between the brackets, face the front of the walker. Grasp the Walker Tray by the sides and clamp the brackets onto the walker's front frame, positioning the lower clamps on top of the walker's lower cross brace.
2. Ensure that the Walker Tray's lower clamps are resting on the walker's lower cross brace, and that all four clamps are securely attached.
3. Pull the Walker Tray out with your thumb and the tips of your fingers; rotate it upward and lock it into place, as shown at right.

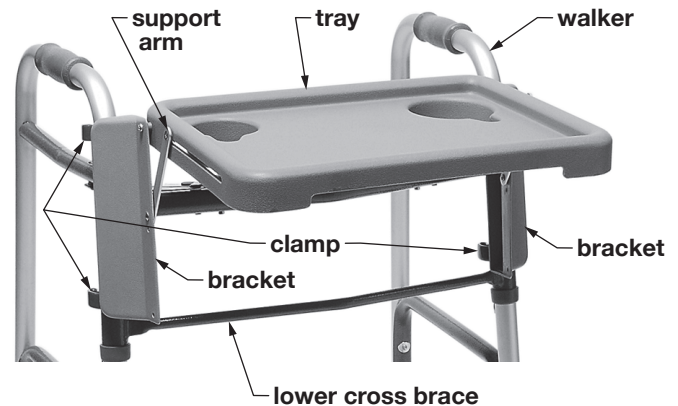603900A Walker Tray

- ⚠ WARNING:** **Ensure that the Walker Tray is securely installed as instructed before use.**
 - 4. When the Walker Tray is no longer in use, lift it up, push in on the tops of both support arms while pushing the tray forward, and rotate the tray down.
- ⚠ WARNING:** **Do not place your hand or fingers between the tray and the bracket.**

WARRANTY

GF Health Products, Inc. warrants the Lumex 603900A Walker Tray for a period of one year for defects in workmanship and materials. During the warranty period, defective items will be repaired or replaced at GF Health Products, Inc.'s option at no charge.

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